



SteraMist[™] Surface Unit Quick Reference Guide

Application:

1. Don appropriate PPE before entering treatment area.
2. Move Surface Unit into treatment space.
3. Open case, remove power cord, and plug unit into a standard wall outlet.
4. Open chemical bottle, remove all safety aluminum bottle covers if present. Insert dip-tube into bottle and secure cap. Place only BIT[™] chemical bottle in bottle compartment with fluid window facing out.
5. Set Prime/Spray switch to Prime. With applicator directed into a (paper cup, sink, waste bin, or chemical bottle) press and hold trigger on applicator until a steady stream of fluid is emitted from nozzle.
6. Release trigger.
7. Switch the Prime/Spray switch to Spray.
8. Apply SteraMist[™] to target surfaces as a light, even mist on desired surfaces. Do Not Over Wet.

Procedure After Use:

1. Remove dip-tube from chemical bottle.
2. With Prime/Spray switch in Spray, apply SteraMist[™] to all parts of the SteraMist[™] Surface Unit (case, underside and wheels, power cord, and hose).
3. Set Prime/Spray switch to Prime, then press applicator trigger to purge remaining chemical from machine into a paper cup, sink, waste bin, or chemical bottle.
4. Place dip-tube into distilled water bottle (no tap water) and press trigger to draw water into machine. After 10 seconds remove dip-tube from water, but keep trigger pressed until water travels through system and out nozzle (approximately 1 minute).
5. Turn power off and unplug machine.
6. Clean inside of applicator nozzle guard, electrodes, posts, nozzle with alcohol prep-pad.
7. Replace applicator and hose to inside of Surface Unit door.
8. Replace power cord to storage compartment inside Surface Unit.
9. Close case. Leave room.